

 **journey**
air **ELITE MAX**

 **journey**
HEALTH & LIFESTYLE



Battery Instruction Guide

Replacing the Battery


1. Turn everything off first.
 - a. Turn off the joystick.
 - b. Turn off the red power switch on the old battery.
 - c. Remove the old battery.
 - d. Install the new battery (the process is the same on both sides).



2. Turn everything back on.
 - a. Turn on the red power switch on the new battery.
 - b. Then turn on the joystick. If the chair works normally, the installation is complete.




If not used for 3 days (72 hours), the battery goes into dormant mode. To use it again, simply turn on the red power switch on the battery.

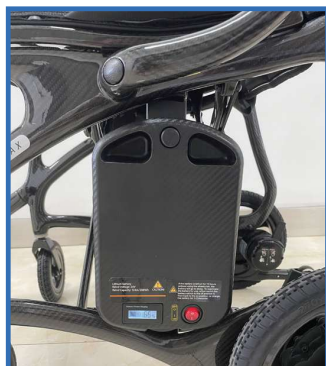

Charging the Battery

1. Before charging, make sure:

- a. Joystick is turned OFF.
- b. Red power switch on the battery is ON.
- c. Wheelchair is in electric mode.
- d. You can charge in two ways:
 - a) Plug the charger into the back of the joystick, then plug into the wall.
 - b) OR remove the battery, plug the charger into the battery's 3-hole port, then plug into the wall.

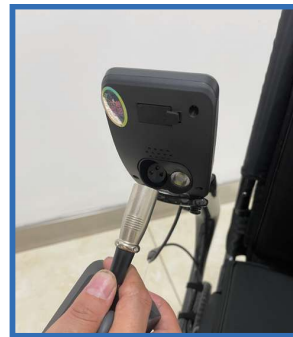


2. Charger lights (indicators):

Orange Light: Power is plugged in.

Blue Light: Charging in progress.

Green Light: Fully charged.

Make sure the blue light is on while charging, and the green light comes on when finished.

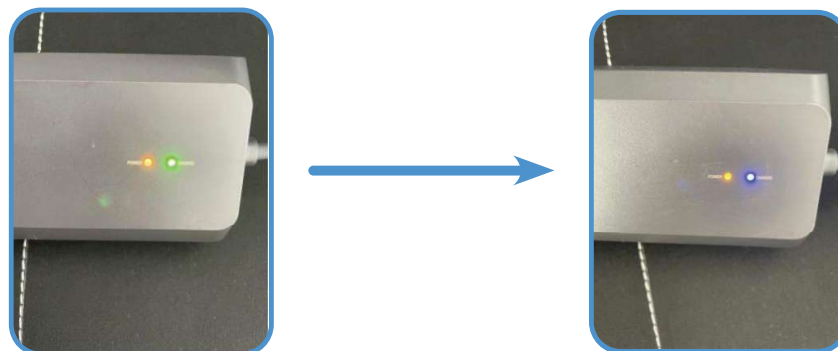

Charging Safety Guidelines

- Charge only in a well-ventilated area.
- Use a 100–240VAC, 50–60Hz, 1.1A power supply.
- Only use the 24V/2A charger provided by our company, which complies with BS EN 60601-1:2006+A13:2024 safety standards.
- Do not use any other chargers. Using unapproved chargers or power sources may cause safety risks or damage.

Following these instructions ensures safe and reliable charging.

Battery Maintenance

- When using the wheelchair for the first time, charge the battery for 6–8 hours before use. This will increase battery performance.
- After the first 4–5 charges, the battery will reach full strength and last longer.
- Always dispose of old batteries according to local recycling or disposal rules.

Battery Safety Instructions

Please read and follow all battery safety instructions before use. Improper use of the battery may cause overheating, fire, leakage, damage, or serious injury.

Do Not

- Put the battery near fire, heaters, or very hot places.
- Drop, crush, or hit the battery.
- Open, puncture, or take apart the battery.
- Let the battery touch metal items (like keys, coins, or jewelry).
- Plug battery directly into a wall socket or car outlet.
- Get the battery wet or put it in water.
- Mix with other batteries (different types or brands).
- Put the battery in a microwave or pressurized container.

Do

Always charge with the provided charger only.

Store the battery at room temperature, in a cool and ventilated place.

If you notice anything unusual (leaking, overheating, bulging, strong smell, or color change), stop using it right away and remove it safely.

If battery fluid gets in your eyes, rinse with clean water immediately and get medical help.

For further guidance, refer to the IEC standards for battery safety.

Quick Reminder:

Turn off the joystick > Switch batteries > Turn new battery on > Charge safely.