

journey SoftSecure

ROTATING SHOWER CHAIR

PRODUCT MANUAL



Scan Here
With Your Phone To
Get Started!



OPEN YOUR
CAMERA



AIM IT AT THE
QR CODE



TAP THE
BANNER

ROTATING SHOWER CHAIR

Now with Microban® Antimicrobial Technology

 journey
HEALTH & LIFESTYLE®

www.shopjourney.com

1.	Introduction and Notes	3
2.	Product Description	4
3.	Microban® Antimicrobial Product Protection	5
4.	Instructions For Use.....	6-8
5.	Specifications and Warranty	9

Welcome to Journey Health & Lifestyle

Thank you for purchasing your SoftSecure Rotating Shower Chair. You've invested in a high-quality, premium materials product that will facilitate comfort and safety.

Safety Instructions

- Read all instructions carefully before using the chair.
- Check all parts for damage and make sure every connection is secure before each use.
- Use this product indoors only.
- Be aware that clothing or skin can be pinched when sitting, standing, or raising and lowering the armrests; move slowly and keep hands clear of moving parts.
- Use extra care on wet or soapy surfaces, as they may be slippery.
- Keep the chair out of reach of children and prevent unauthorized use.
- Do not exceed the 300 lbs maximum user weight listed for this chair.

2. PRODUCT DESCRIPTION

Parts of the Product

TREATED WITH
MICROBAN



-  1. Backrest
-  2. Seat
- 3. Flip Up Armrest
- 4. Rotating Plate with Knob
- 5. Push Button

3. MICROBAN® ANTIMICROBIAL PRODUCT PROTECTION

Microban® Antimicrobial Product Protection

- Microban® antimicrobial* product protection is built-in to make your Rotating Shower Chair last longer and look cleaner
- Microban® antimicrobial* product protection helps prevent the uncontrolled growth of bacteria, mold and mildew on the Rotating Shower Chair and provides 24/7 cleanliness protection



Microban® is a registered trademark of Microban Products Company

*** These antimicrobial properties are built-in to protect the Rotating Shower Chair. The Rotating Shower Chair does not protect users or others against bacteria, viruses, germs or other disease organisms**

4. INSTRUCTIONS FOR USE

Assembly

Step 1: Attach the Legs

Press the push buttons on each leg and slide the legs into the tubes on the frame until the buttons “snap” through the holes. Confirm that all legs are adjusted to the same height and the chair stands steady and level before use.



4. INSTRUCTIONS FOR USE

Step 2: Attach the Backrest

Press the push buttons on the backrest and slide the backrest posts into the tubes located on each side and at the back of the seat. Release the buttons and gently pull up on the backrest to be sure it is locked securely.



Step 3: Final Stability Check

Before using the chair, make sure the seat is level and does not rock. If needed, readjust the leg height so the chair feels stable and secure.

4. INSTRUCTIONS FOR USE



Rotating Seat

1. Pull the knob at the rotating mechanism and slowly turn the seat to face the user. Keep turning until the seat “clicks” and will not move further, then release the knob to lock it in position.
2. To unlock and rotate again, pull the knob out and turn the seat to the desired position. Release the knob and gently try to move the seat to confirm it is locked.
3. When the user is seated, always position the seat so it is locked and does not rotate freely before standing or transferring.
4. Lock the chair when the user is facing the open side of the bath or shower so they can stand up slowly and safely.
5. To flip up an armrest, press or push the button on the armrest, then lift the armrest gently until it reaches the upright position.



Product Specifications

Product Dimension	(18"-19") x (18"-19") x (30"-34")
Weight Capacity	300 lbs
Packaging Dimensions	25" x 18" x 16"
Net Weight	21 lbs
Material on Product	Aluminum

Warranty

Journey Health & Lifestyle warrants the SoftSecure Rotating Shower Chair frame to be free from defects in materials, workmanship and assembly for twelve (12) months from the original purchase date. The warranty does not extend to non-durable components such as the rubber tips.



 **journey**
SoftSecure

ROTATING SHOWER CHAIR

If you have any questions, call us at our toll free number:

1-800-958-8324