

# journey SoftSecure

COMMODE WITH BACKREST

## PRODUCT MANUAL



**Scan Here**  
With Your Phone To  
**Get Started!**



OPEN YOUR  
CAMERA



AIM IT AT THE QR  
CODE



TAP THE  
BANNER

COMMODE WITH BACKREST

Now with Microban® Antimicrobial Technology

 journey  
HEALTH & LIFESTYLE®

[www.shopjourney.com](http://www.shopjourney.com)

<b>1.</b> Introduction and Notes .....	3
<b>2.</b> Product Description .....	4
<b>3.</b> Microban® Antimicrobial Product Protection .....	5
<b>4.</b> Instructions For Use.....	6-8
<b>5.</b> Specifications and Warranty .....	9

## Welcome to Journey Health & Lifestyle

Thank you for purchasing your SoftSecure Commode with Backrest. You've invested in a high-quality, premium materials commode with backrest that will facilitate comfort and safety.

## Safety Instructions

- Read these instruction for use carefully.
- All components should be checked for damage and a secure fit prior to use.
- Approved for indoor use only.
- Clothing or body parts can be pinched when sitting down, standing up, or inserting the toilet bucket.
- Prevent unauthorized use, for example, by children.
- Maximum weight capacity 300 lbs.

### Parts of the Product



1. Armrest
2. Seat
3. Height Adjustable Leg
4. Rubber Tip
5. Backrest
6. Commode Bucket

#### Microban® Antimicrobial Product Protection

- Microban® antimicrobial\* product protection is built-in to make your Commode With Backrest last longer and look cleaner
- Microban® antimicrobial\* product protection helps prevent the uncontrolled growth of bacteria, mold and mildew on the Commode With Backrest and provides 24/7 cleanliness protection



Microban® is a registered trademark of Microban Products Company


**\* These antimicrobial properties are built-in to protect the Commode With Backrest. The Commode With Backrest does not protect users or others against bacteria, viruses, germs or other disease organisms**

## Assembly Instructions

### Step 1: Attach the Armrests

Align each armrest with the open tubes on both sides of the seat. Slide the armrest tubing into place, then tighten the knobs by hand until secure. Make sure both armrests are firmly attached and evenly positioned.



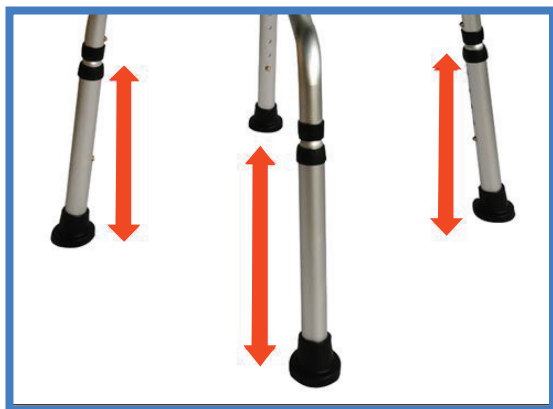

### Step 2: Attach the Legs

Press the push buttons on each leg while sliding them into the tubes on the frame. Continue sliding until you hear or feel the push buttons “snap” through the holes.

Check that:

- Each button is fully engaged and visible through the hole.
- All four legs are adjusted to the **same height** so the bench sits level and stable.

### Step 3: Attach the Backrest

Insert the backrest tube into the openings on the rear of the seat frame. Push down gently until the backrest locks securely into place. Confirm that it is steady before use.



### **Insert the Commode Bucket**

From the front of the seat, slide the commode bucket carefully into the guide rails located underneath. Push until it clicks or stops in position. When removing for cleaning, pull the bucket straight out along the same guide rails.



## Product Specifications

<b>Product Dimension</b>	(26"-27") x 18" x (31"-35")
<b>Weight Capacity</b>	300 lbs
<b>Packaging Dimensions</b>	22" x 10" x 25"
<b>Net Weight</b>	20 lbs
<b>Material on Product</b>	Aluminum

## Warranty

Journey Health & Lifestyle warrants the SoftSecure Commode With Backrest frame to be free from defects in materials, workmanship assembly for twelve (12) months from the original purchase date. The warranty does not extend to non-durable components such as the rubber tips.



# journey SoftSecure

COMMODE WITH BACKREST

If you have any questions, call us at our toll free number:

**1-800-958-8324**