

journey SoftSecure

ROTATING TRANSFER BENCH

PRODUCT MANUAL



ROTATING TRANSFER BENCH

 journey
HEALTH & LIFESTYLE®

www.shopjourney.com

1. Introduction and Notes	3
2. Product Description	4
3. Microban® Antimicrobial Product Protection	5
4. Instructions For Use.....	6-8
5. Specifications and Warranty	9

Revision Date: February 2026

Welcome to Journey Health & Lifestyle

Thank you for purchasing your SoftSecure Rotating Transfer Bench. You've invested in a high-quality, premium materials rotating bench that will facilitate comfort, safety and functionality while you get in and out of your tub.

Safety Instructions

- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use.
- Approved for indoor use only.
- Clothing or body parts can be pinched when sitting down or standing up, as well as when raising the armrests and locking them into place.
- Prevent unauthorized use, for example, by children.
- Maximum weight capacity 300 lbs.

Parts of the Product



Microban® Antimicrobial Product Protection

- Microban® antimicrobial* product protection is built-in to make your Rotating Transfer Bench last longer and look cleaner
- Microban® antimicrobial* product protection helps prevent the uncontrolled growth of bacteria, mold and mildew on the Rotating Transfer Bench and provides 24/7 cleanliness protection



Assembly Instructions

Step 1: Attach the Side Frames

Slide the side frame tubing into the main section receivers. Secure each side using the provided knobs.


Important: Make sure the holes on the main frame face outward so the sliding mechanism can function properly. **Do not cover these holes;** doing so will prevent the seat from sliding.





Step 2: Install the Backrest

Press the push buttons on each side of the backrest and gently slide the backrest posts into the receivers located at the rear sides of the seat. Once in place, release the buttons to lock securely.

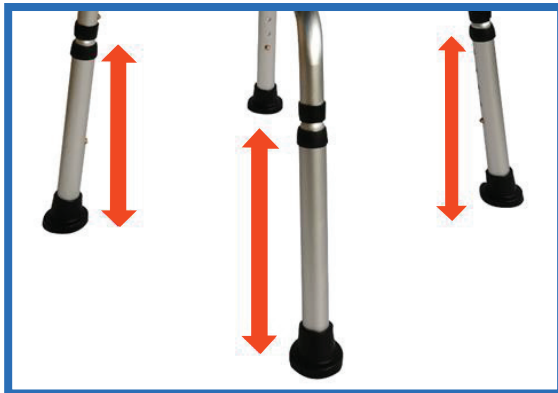

Step 3: Adjust the Legs

Set the bench height to your comfort level by pressing the adjustment buttons and sliding each leg to the desired height. Make sure all four legs are set evenly before use for stability.



Step 4: Flip Up the Armrest

To raise the armrest, press the button located on the side of the seat and lift gently until it clicks into the upright position.

Rotate the Seat

1. To turn the seat, pull out the knob located at the rotation mechanism.
2. Rotate the seat slowly until it locks into your preferred position.
3. To unlock, pull the knob again to adjust or return the seat to its original position.
Always make sure the seat is locked firmly before sitting or transferring.



Slide the Seat Side-to-Side

To move the seat along the frame, pull out the sliding knob. Once the seat reaches the desired position, release the knob to lock it securely in place.



Product Specifications

Product Dimension	(30"-34") x (19"-20") x (32"-36")
Weight Capacity	300 lbs
Packaging Dimensions	27" x 18" x 12"
Net Weight	25 lbs
Material on Product	Aluminum

Warranty

Journey Health & Lifestyle warrants the SoftSecure Rotating Transfer Bench frame to be free from defects in materials, workmanship and assembly for twelve (12) months from the original purchase date. The warranty does not extend to non-durable components such as the rubber tips.



journey SoftSecure

ROTATING TRANSFER BENCH

If you have any questions, call us at our toll free number:

1-800-958-8324