

journey SoftSecure

UPLIFT COMMODE

PRODUCT MANUAL



Scan Here
With Your Phone To
Get Started!



UPLIFT COMMODE

 journey
HEALTH & LIFESTYLE®

www.shopjourney.com



OPEN YOUR
CAMERA

AIM IT AT THE QR
CODE

TAP THE
BANNER

1. Introduction and Notes	3
2. Product Description	4
3. Microban® Antimicrobial Product Protection	5
4. Instructions For Use.....	6
5. Specifications	7

Revision Date: February 2026

Welcome to Journey Health & Lifestyle

Thank you for purchasing your SoftSecure Uplift Commode. You've invested in a high-quality, premium materials commode that will actively help you stand up and will facilitate comfort and safety.

Safety Instructions

- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use.
- Approved for indoor use only.
- Clothing or body parts can be pinched when sitting down, standing up, inserting the toilet bucket, as well as when raising the armrests and locking them into place.
- Prevent unauthorized use, for example, by children.
- Maximum weight capacity 300 lbs.

Parts of the Product



- 1. Backrest
- 2. Armrest
- 3. Seat
- 4. Leg Tube
- 5. Rubber Tip
- 6. Commode Bucket

Microban® Antimicrobial Product Protection

- Microban® antimicrobial* product protection is built-in to make your Uplift Commode last longer and look cleaner
- Microban® antimicrobial* product protection helps prevent the uncontrolled growth of bacteria, mold and mildew on the Uplift Commode and provides 24/7 cleanliness protection



Microban® is a registered trademark of Microban Products Company

*** These antimicrobial properties are built-in to protect the Uplift Commode. The Uplift Commode does not protect users or others against bacteria, viruses, germs or other disease organisms**

Assembly

Step 1: Attach the Backrest

Press the push buttons on the backrest tube and slide the tube into the openings located under the seat. Make sure the backrest feels secure and does not wobble.



Step 2: Attach the Legs

Press the push buttons on each leg while sliding the legs into the tubes on the frame. Continue until each button clicks through the hole. Check that all buttons are visible in the holes, the legs feel stable, and all legs are adjusted to the same height so the chair sits level.



Step 3: Insert the Commode Bucket

From the front of the seat, slide the commode bucket into the guide rails located underneath the seat. Push gently until it stops and sits securely in place.



Product Specifications

Product Dimension	(30"-31") x (23"-24") x (34"-38")
Weight Capacity	300 lbs
Packaging Dimensions	25" x 17" x 22"
Net Weight	16 lbs
Material on Product	Aluminum

Warranty

Journey Health & Lifestyle warrants the SoftSecure Uplift Commode frame to be free from defects in materials, workmanship and assembly for twelve (12) months from the original purchase date. The warranty does not extend to non-durable components such as the rubber tips.



journey

SoftSecure

UPLIFT COMMODE

If you have any questions, call us at our toll free number:

1-800-958-8324